

Medical Gas System Maintenance Program

Routine preventative maintenance (PM) is required to help reduce costly failures, leaks and unnecessary downtime. An Ohio Medical System Maintenance Program is an excellent and cost effective way to make sure that routine preventative maintenance is performed on all of your medical gas equipment.

Our history in the medical gas industry includes over 40 years of system manufacturing and 15,000+ medical gas pumping systems installed world-wide.

We have fabricated systems for most of the major brand name manufacturers as well as produced and sold our own HEALTHCAIR® line.

We are pleased to offer a Medical Gas System Maintenance Program for your facility from Ohio Medical. We will conduct a system assessment to determine the present condition of your equipment and to establish a benchmark for future preventative maintenance.

Program Participants Receive the Following Program Benefits:

- Preventative maintenance service will be conducted during normal workday, work hours, at specified intervals
- A 10% discount off list price on any repair parts for systems under the contract, purchased during the term of the contract
- Guaranteed priority service for repairs outside the standard PM, during the term of the contract, at a 10% discount from our standard rate schedule

Periodic (quarterly or semi-annually) PM including each of these activities:

Medical Air Packages

- Check/change intake filters per the manufacturer's guidelines
- Evaluate system condition and recommend repairs or upgrades
- Check and evaluate Dew Point and CO monitoring equipment
- Check and test alarms and alarm set points

Medical Air Dryer Systems

- Evaluate dryer effectiveness
- Change filters per manufacturer's guidelines

Medical Vacuum Systems

- Check/change oil and filters per the manufacturer's guidelines
- Evaluate system condition and recommend repairs or upgrades
- Check and test alarms and alarm set points

Medical Gas Manifolds

- Check for leaks
- Verify proper operation
- Verify line pressure
- Adjust primary and secondary regulators

Medical Gas Alarms

- Test audible alarms
- Verify and adjust visible alarms
- Verify and adjust pressure settings



Prior to commencing any repairs beyond the scope of the medical gas system maintenance program, as outlined, we must receive your written approval to proceed based upon our recommendations and estimates.