


Sure-Trap™ Overflow Safety Trap

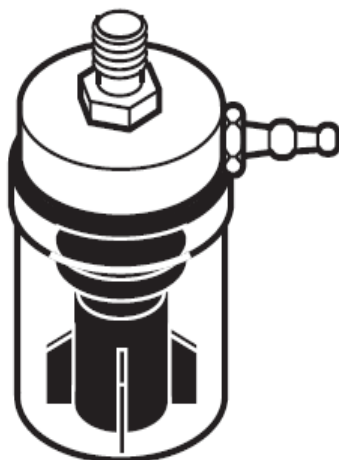

Rx Only

Instructions for Use

Safety Instructions

This manual provides you with important information about the Sure-Trap Overflow Safety Trap. To ensure the safe and proper use of this device, read and understand all of the safety and operating instructions. If you do not understand these instructions, or have any questions, refer to service manual, contact your supervisor, dealer, or the manufacturer before attempting to use the device.

Intended Use

The Ohio Medical Sure-Trap Overflow Safety Trap is designed to prevent fluids from entering the vacuum regulator or central suction system.

Receiving/Inspection

Remove product from package and inspect for damage. If product is damaged, do not use and contact your dealer or equipment provider.






User Responsibility

This Product will perform as described in these instructions when installed and operated in accordance with the instructions provided.

WARNING

- ⚠ Do not use this device in the presence of flammable anesthetics.
- ⚠ This device is to be used only by persons who have been adequately instructed in its use.
- ⚠ Not for field or transport use.

Definitions

WARNING:	Possibility of injury to the patient or the operator exists.		
CAUTION:	Possibility of damage to the equipment exists.		
Rx Only	Caution: Federal (USA) Law restricts this device to sale by or on the order of a licensed healthcare provider.		
	Warnings and Cautions		Manufacturer
	Catalog Number		Manufacturer's Batch Code
	Consult instructions for use		

Specifications

Height = Approximately 4 inches

Width = Approximately 2 inches

Vacuum:

0 to 650 mm Hg

Operating Temperature Range:

39.2°F to 120.2°F (4°C to 49°C)

Storage Temperature Range:

-0.4°F to 159.8°F (-18°C to 71°C)

Operating and Storage Relative Humidity:

5 to 95%




MR Safe

Product Setup:

CAUTION

- ⚠ This device must be in the vertical position to function.
- ⚠ Before each use, inspect the device for any damage; DO NOT USE if damaged or visibly contaminated.
- ⚠ Do not over-tighten the Sure-Trap to vacuum regulator.
- ⚠ Conduct visual pre-use check to ensure the deflector shield is attached to the cap.

Attach the Sure-Trap to inlet of Vacuum Regulator and attach suction tubing from the barb fitting to the vacuum port of collection canister as shown.

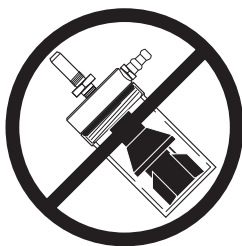

Cleaning

The exterior of the device may be cleaned with mild cleaning or disinfecting solution.

Disposal

WARNING

- ⚠ After patient use, suction equipment may be contaminated.
- If liquid is present in the device, remove from service and discard immediately.
- ⚠ Dispose of trap in accordance with the local regulations using biohazard precautions.

Replacement Part Numbers

Box of 12 Sure-Traps: 7700-0365-912

Box of 25 Sure-Traps: 7700-0365-925

Box of 50 Sure-Traps: 7700-0365-950

Troubleshooting

Problem	Probable Cause	Remedy
No Suction	1. Leaking Connections 2. Suction Source not turned "ON" 3. Trap Full	1. Check connections and reconnect 2. Turn Suction Source "ON" 3. Replace Trap
Fluid Exits Bottle	1. Deflector Loose 2. Crack in Bottle	1. Replace Trap
Cloudy/Opaque Bottle	1. Ingress of dirt/debris	1. Replace Trap



Ohio Medical, LLC
1111 Lakeside Drive
Gurnee, IL 60031 USA
P: +1 866 549 6446
www.ohiomedical.com

© 2017 Ohio Medical, LLC.

This document contains information that is proprietary and confidential to Ohio Medical, LLC.

Use of this information is under license from Ohio Medical, LLC.

Any use other than that authorized by Ohio Medical, LLC is prohibited.

Ohio Medical and the Ohio Medical Logo are registered trademarks of Ohio Medical, LLC.