

# Push-To-Set™ Digital Intermittent Suction Unit (ISO/CE) Pediatric/Neonatal

## DESCRIPTION AND APPLICATION

The Ohio Medical® Pediatric and Neonatal Push-To-Set™ Digital Intermittent Suction Unit is a medical suction device designed to deliver continuous or intermittent controlled vacuum from a hospital pipeline vacuum system. It provides the versatility of an intermittent mode for nasogastric drainage and a continuous mode for tracheal, pharyngeal, or surgical suctioning. The Pediatric and Neonatal model is designed to prevent vacuum adjustment above a preset vacuum limit level that may be unsafe. This device is ideal for neonatal and pediatric patients or sensitive applications where limiting the peak vacuum level is important. The intermittent mode uses a pneumatic logic device to turn the suction “ON” and “OFF” automatically. This unique Unilogic timing module sets the industry standard for quiet, reliable, controlled intermittent suction. The Pediatric model is adjustable from 0-18.0 kPa (0-135 mmHg) and the Neonatal model is adjustable from 0-13.3 kPa (0-100 mmHg), to meet most low clinical suction requirements. It also contains a positive pressure relief to reduce potential injury to the patient and equipment if momentarily attached to a positive pressure gas source.

The Pediatric and Neonatal Push-To-Set™ Intermittent Suction Units are a low vacuum/high flow regulator.

## FEATURES AND BENEFITS

- **EXCLUSIVE! Patented Push-To-Set™**  
Improves patient safety  
May prevent inadvertent unregulated suction  
Simple, one handed operation
- **Baby Icon indicates Pediatric or Neonatal Limited Model**
- **Negative Pressure Relief Valve**  
Pediatric model is factory preset to 18.0 kPa (135 mmHg)  
Neonatal model is factory preset to 13.3 kPa (100 mmHg)
- **Unilogic Timing Module**  
Pneumatic, quiet, reliable operation  
Returns to atmospheric pressure during “OFF” cycle  
Timing cycles  
Independently adjustable 3 to 30 seconds “ON” and “OFF” cycles  
Factory preset at 15 seconds on/8 seconds off (± 3 seconds)  
Begins in the “ON” mode
- **Flow Rates**  
(Intermittent) Preset at 8 L/min (ASTM)  
(Continuous) 0 to 80 L/min (without fittings at full increase)
- **Large, Easy to Read Gauge**  
Color coded ranges  
Unmatched accuracy for greater safety: ± 1% of full scale
- **Safety Features**



Digital Push-To-Set™ Pediatric Intermittent Suction Unit



Digital Push-To-Set™ Neonatal Intermittent Suction Unit

Push-to-Set™ automatic occlusion design  
Positive pressure safety relief valve

- **Modular Components**  
Common components reduce service part inventory  
Simplifies service procedures  
Low cost replacement parts and kits
- **Durable Design and Construction**  
Few mechanical parts  
Rugged, shatter-resistant case  
Corrosion & Lubrication Free  
Service-free backplate
- **CE marked**
- **Ordering Options**  
Available in various configurations including adapters for connection to source vacuum. Also available with options for collection devices and overflow safety protection.

## TECHNICAL SPECIFICATIONS\*

### Flow Rate

Intermittent – 0-13 L/min (preset at 8 L/min per ASTM)  
 Continuous – 0-80 L/min (not adjustable) without fittings  
 at full increase

### Timing

Easily adjustable silent ON/OFF  
 Factory setting Intermittent Timing: 15 seconds ON/8 seconds  
 OFF (±3 seconds)  
 Begins in the “ON” mode

### Vacuum Range

0-18.0 kPa (0-135 mmHg) Pediatric  
 0-13.3 kPa (0-100 mmHg) Neonatal

### Gauge Range

0-21.3 kPa (0-160 mmHg) Pediatric  
 0-13.3 kPa (0-100 mmHg) Neonatal

### Gauge Accuracy

±1% of Full Scale (1.6 mmHg) Pediatric  
 ±1% of Full Scale (1.0 mmHg) Neonatal

### Dimensions

16.5 x 7.1 x 12.2 cm

(6.5" H X 2.8" W X 4.8" D)

**Weight:** 0.57 Kg (1 lb 4oz) less fittings and adapters

**Battery:** Primary Lithium: 2/3AA, 3.6V, 1.6 Ah

**F.O.B. Point:** Gurnee, IL

**Applied Standards:** ASTM F960 and ISO 10079-3

\* Specifications are nominal, subject to change without notice. Please see "Instructions for Use" manual for the most current regulated vacuum ranges.

## SHIPPING INFORMATION

### Shipping Weight

0.77 Kg (1 lb 11 oz) (less fittings and adapters)

### Package Size

16.5 x 26.67 x 20.32 cm  
 (6.5" H x 10.5" W x 8" D )

### Packaging

One per carton

## Parts Configuration

**87XX - XXXX - 9XX**

**Push-To-Set™ Series**

### Adapter (vacuum source connection)

No Adapter with 1/8" NPT Female.....	00
BOC Type (British Standard).....	31
DIN Type (German Standard).....	32
AFNOR Type (French Standard).....	33
AGA Type (Nordic Standard).....	34

### Fittings (patient connection)

No Fitting 1/8" NPT Female.....	00
Tubbing Nipple.....	01
Locking Gland.....	04
Locking Gland & Overflow Safety Trap.....	05
Locking Gland, Overflow Safety Trap, 1300 ml Plastic Bottle with Wall Slide, Tubing.....	06

### Push-To-Set™ Digital Intermittent Suction Unit

#### Pediatric

6373	Off-White*
6Y73	Yellow
6B73	Blue
6L73	Pink
6R73	Red
6L73	Lt. Blue
6U73	Purple

#### Neonatal

6372	Off-White*
6Y72	Yellow
6B72	Blue
6K72	Pink
6R72	Red
6L72	Lt. Blue
6U72	Purple

\*Standard Color



**EMERGO EUROPE**  
 Prinsessegracht 20 2514 AP The Hague The Netherlands

**AUSTRALIAN SPONSOR: EMERGO AUSTRALIA**  
 Level 20, Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia

© 2019 Ohio Medical LLC. All rights reserved. This document contains information that is proprietary and confidential to Ohio Medical LLC. Use of this information is under license from Ohio Medical LLC. Any use other than that authorized by Ohio Medical LLC is prohibited. Ohio Medical and the Ohio Medical logo are registered trademarks and Push-To-Set is a trademark of Ohio Medical LLC. U.S. Patent #6,623,463 (Push-To-Set™ Vacuum System).



255452 (Rev. 6.1) 08/19

1111 Lakeside Drive Gurnee, IL 60031 USA  
 866.549.6446 | ohio-medical.com