

Operating Quick Guide

instavac[®] I
MOBILE ASPIRATOR



1. Connect tubing with filter to the tube nipple at top of the pole and to the canister Vacuum port.
2. Attach patient tubing to canister Patient port.
3. Verify presence of clean Overflow Safety Trap and that all tubing connections are secure.
4. Plug in power cord to AC electrical outlet.
5. Push Suction Mode Switch to CONSTANT.
6. Turn the device on by pushing Power Switch to ON (I) position.
7. Verify presence of suction with your fingertip at the end of the patient tubing.
8. Pinch or occlude patient tubing and turn the Vacuum Control Knob to adjust suction to the desired level.
9. If needed, Push Suction Mode Switch to INTERMITTENT and verify that suction starts and stops.

Refer to manual for reprocessing steps between patients. For full instructions, precautions and specifications refer to the operating manual ([550102](#)).