

SUCTION FILTER With 1/4" Adapter Quantity: 20 Each

SUGGESTED PROCEDURE:

- ◆ Attach the filter's tubing adapter to the suction (vacuum) port to the collection canister lid.
- ◆ Attach the other filter port to the vacuum source.

WARNING:

- ◆ Connection of the tubing adapter to the other non-rigid tubing could cause obstruction of the tubing resulting in premature canister shut-off.

CAUTION:

- ◆ For single patient use only. Reuse can cause cross infections. Use of sterilization or cleaning solution may dangerously increase resistance and/or result in retention of harmful residuals.
- ◆ Federal law in the U.S.A. and Canada restricts this device to sale by, or on the order of, a licensed physician only.

Re-order Part Number: 57888001

550016 (Rev.4) 07/05