

Intermittent Suction Unit Pediatric

DESCRIPTION AND APPLICATION

The Ohio Medical® Pediatric Intermittent Suction Unit (PED-ISU) is a medical suction device designed to deliver continuous or intermittent controlled vacuum from a hospital pipeline vacuum system. It provides the versatility of an intermittent mode for nasogastric drainage and a continuous mode for tracheal, pharyngeal, or surgical suctioning. The Pediatric model is designed to prevent vacuum adjustment above a preset vacuum limit level. This device is ideal for neonatal and pediatric patients or sensitive applications where limiting the peak vacuum level is important. The intermittent mode uses a pneumatic logic device to turn the suction “ON” and “OFF” automatically. This unique Unilogic Module provides quiet and reliable controlled intermittent suction. The Pediatric model is adjustable from 0-150 mmHg, to meet most low clinical suction requirements. It also contains a positive pressure relief valve to reduce potential injury to the patient and equipment if momentarily attached to a positive pressure gas source.

The Pediatric Intermittent Suction Unit is a low vacuum/high flow regulator.



Pediatric Intermittent Suction Unit

FEATURES AND BENEFITS

- **Baby Icon indicates Pediatric/Limited Model**
- **Vacuum Control**
Fully adjustable vacuum level from 0-150 mmHg
Peak vacuum limit adjustable from 80-150 mmHg,
factory preset to 135 mmHg
- **Unilogic Timing Module**
Pneumatic, quiet, reliable operation
Returns to atmospheric pressure during “OFF” cycle
Timing cycles
 - Independently adjustable 3 to 30 seconds “ON” and “OFF” cycles
 - Factory preset at 15 seconds on/8 seconds off (begins in the off mode) Flow rates
- **Flow Rates**
Intermittent: 0 to 13 L/min, preset at 8 L/min (ASTM)
Continuous: 0 to 80 L/min (not adjustable)
- **Large, Easy to Read Gauge**
5 mmHg increments
Accuracy ± 5% full scale
- **Positive pressure and safety relief valve**
- **Modular Components**
Simplifies service procedures
Low cost replacement parts and kits
- **Durable Design and Construction**
Few mechanical parts
Rugged, shatter-resistant case
- **CE marked**
- **Ordering Options**
Available in various configurations to include adapters for connection to source vacuum. Also available with options for collection devices and overflow safety protection.

SPECIFICATIONS*

Flow Rate

Intermittent – 0-13 L/min (preset at 8 L/min per ASTM)
 Continuous – 0-80 L/min (not adjustable) without fittings at full increase

Timing

3 to 30 seconds
 Independent on and off
 Preset 15 seconds on/8 seconds off
 Begins in the “OFF” mode

Vacuum Range

0-20.0 kPa (0-150 mmHg)

Gauge Range

0-21.3 kPa (0-160 mmHg)

Gauge Accuracy

±10 mmHg or ±5% Full Scale Deflection

Dimensions

6.6” H x 3.5” W x 4.8” D
 (16.8 x 8.9 x 12.2 cm)

Weight

1 lb 12oz (0.793 Kg)

Applied Standards

ISO 10079-3

* Specifications are nominal, subject to change without notice.

SHIPPING INFORMATION

Shipping Weight

2 lbs 8 oz (1.1 Kg) less fittings and adapters


Package Size

8” H x 3.5” W x 4.7” D
 (20.3 x 8.9 x 12 cm)

Packaging

One per carton

Parts Configuration

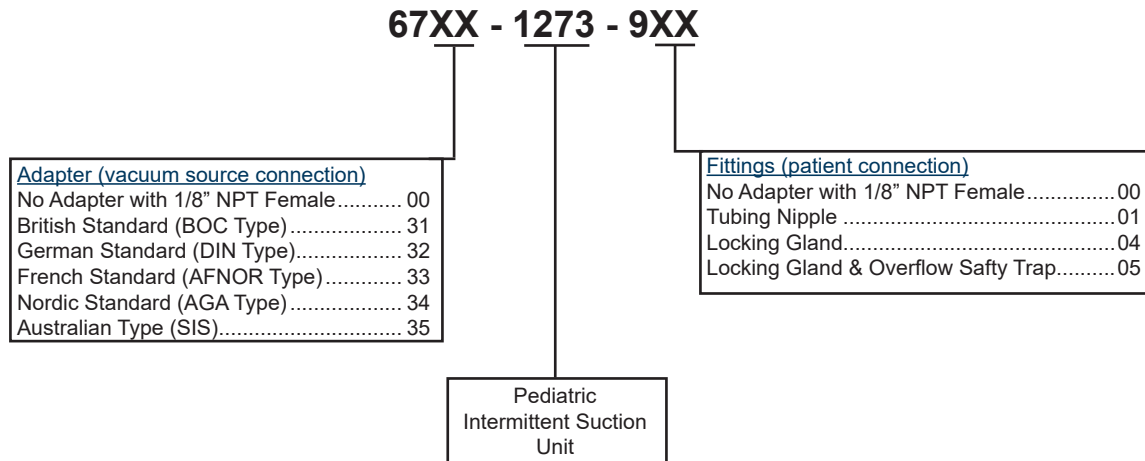

© 2018 Ohio Medical LLC.

This document contains information that is proprietary and confidential to Ohio Medical LLC. Use of this information is under license from Ohio Medical LLC. Any use other than that authorized by Ohio Medical LLC is prohibited.

Ohio Medical and the Ohio Medical logo are registered trademarks of Ohio Medical LLC.



EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands

Ohio Medical LLC
 1111 Lakeside Drive
 Gurnee, IL 60031 USA
 866.549.6446 | ohio-medical.com
 550042-EN-ISU-PED (Rev. 3) 11/18