

# Medical Oxygen Humidifier

## DESCRIPTION AND APPLICATION

Ohio Medical's Oxygen Humidifier is a non-disposable unit designed for durability and performance in the hospital or at home. It is used to provide moisture to oxygen administered to the patient using the bubble-type humidification process. The dishwasher-safe jar and lid are made of shatter-resistant, durable plastics that can be steam or gas sterilized.

The jar is lightweight and compact for better clearance of hospital shelves, beds and gas cylinders. The maximum and minimum water lines are boldly labeled. The jar holds 315 mL of water at the max line. The lid assembly contains a pressure relief valve, an O2 DISS female wing nut, a diffuser head and a tubing nipple outlet. The all metal DISS connection provides strength and durability. If the patient tubing becomes occluded and the positive pressure reaches 2 PSI, the pressure relief valve will set off an alarm. The diffuser head provides small bubbles for quiet operation with high humidity output, and may be detached for easy cleaning.

## TECHNICAL SPECIFICATIONS

### Product Dimensions (Diameter x Height)

3.25 x 7.5 in/82.5 x 190 mm

### Safety Relief Valve

2 psi minimum with audible alarm

### Jar Capacity to max line

315 mL

### Inlet Fitting

Oxygen DISS female with metal nut

### Outlet Fitting

Hose barb for 3/16" to 1/4" inner diameter plastic tubing

\*Specifications are nominal, subject to change without notice.

## SHIPPING INFORMATION

**Net Shipping Weight:** 10 oz/0.28 kg

**Package Dimension (Height x Width x Length):** 8 x 3.75 x 4.3 in/203 x 95 x 110 mm

**Warranty:** One Year

## ORDERING INFORMATION

**Humidifier with Oxygen DISS Connection:** 6700-0338-800

This product meets requirements of The Medical Device Directive (CE).



© 2016 Ohio Medical LLC.

This document contains information that is proprietary and confidential to Ohio Medical LLC. Use of this information is under license from Ohio Medical LLC. Any use other than that authorized by Ohio Medical LLC is prohibited. Ohio Medical and the Ohio logo are registered trademarks of Ohio Medical LLC.

TRUSTED BRANDS OF OHIO® MEDICAL

aeros®

AMVEX

Ohio®

Squire-Cogswell

Ohio Medical - 1111 Lakeside Drive - Gurnee, IL 60031-4099 USA  
Toll Free: 866-549-6446 - Fax: 847-855-6218 - www.ohiomedical.com  
Form No. SOT108 (Rev. 8) 05/2016